**Phantom Series / Phantom-Serie /
Phantom Serie / Série Phantom /
Serie Phantom / Serie Phantom /
Série Phantom**

AL Retractor System / AL-Retraktor-System /
AL retractiesysteem / Sistema Retrator AL /
Sistema retractor AL / Sistema di divaricazione AL /
Système d'écarteur AL

AL-1000

USER GUIDE / GEBRAUCHSANWEISUNG /
GEBRUIKERSHANDLEIDING /
GUIA DE UTILIZADOR / GUÍA DEL USUARIO /
MANUALE UTENTE / GUIDE D'UTILISATION

TeDan Surgical Innovations, Inc.
12320 Cardinal Meadow Dr.
Suite 150
Sugar Land, TX 77478 USA
P: 713-726-0886
F: 713-726-0846
info@tedansurgical.com

TeDan Surgical Innovations, B.V.
Kantstraat 19
NL-5076 NP Haaren
The Netherlands
P: +31 (411) 623791 (EU)
EAR@tedansurgical.com

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Phantom AL Retractor System Setup

Arm Assembly:

- Loosen knob of **Rotating Table Clamp** (ML-0021). Attach table clamp to the surgical rail over the sterile drape by the top and bottom jaws. Tighten knob to secure clamp at the desired location. Using the same method, attach the second rotating table clamp to the opposite side of the table.

Note: Table clamps should be positioned so that one table clamp is positioned as far caudally while the other is positioned as far cranially as possible while still allowing the arm clamp to attach to the ring.

- Insert an **Articulating Arm** (AL-0110) into each of the **Rotating Table Clamps**. Position the **Articulating Arm and Table Clamp** so that they are angled as low as possible and will not impede the surgeon's movement (Figure One).

Note: When loosening, do not force the knob of the articulating arm past the stop. Doing so could damage the ball joint and affect the rigidity of the articulating arm.

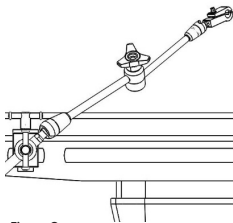
Figure One
Rotating Table Clamp and Articulating Arms attached to operating table

Retractor Ring Assembly:

- Clear enough space on a flat surface for assembly of the **Retractor Ring** (AL-0100). Each of the following **Retractor Ring** steps must be conducted on a flat surface to ensure that the screws and forks are properly aligned.
- The **Retractor Ring** consists of two ring segments. To loosen the screws,

turn the screws counterclockwise no more than two times on each ring segment using the **Hex Tool** (ML-0505). Assemble the **Retractor Ring** by pushing the two segments together.

Note: Connection points of the Retractor Ring should be brought to edge of flat surface to provide clearance for Hex Tool during assembly.

- With the **Hex Tool**, tighten each screw on the **Retractor Ring** to secure the assembly (Figure Two & Figure Three).

Note: To prevent damage to the screws and forks on the connection points of the Retractor Ring, pull rings directly apart. Do not cant or twist the rings.

Retractor Ring to Articulating Arms Assembly:

- Using the **Hex Wrench** (AL-0106), unscrew the ring attachment of both **Articulating Arms**. Insert the **Retractor Ring** into the clamps and tighten the clamps around the ring where marked "Attach Rigid Arm Here" (Figure Four).
- Adjust the **Articulating Arms** to position the **Retractor Ring** in accordance with the markings that indicate the Caudal and Cephalic direction. To prevent movement of the **Retractor Ring**, ensure that the knob of the **Articulating Arms** are completely tightened.

Note: If a larger frame is desired, Phantom Extension Bars (AL-0101) may be attached to the Ring Segments (AL-0100) to create an ovalar retractor ring.

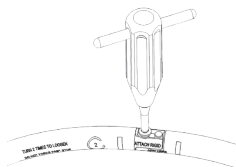
Figure Two
Hex Tool tightening Retractor Ring assembly

Note: Do not force the screw past the stop. Excessive force may damage the assembly mechanism.

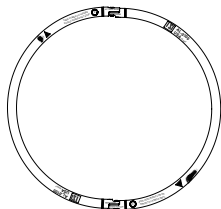
Figure Three
Retractor Ring Assembly

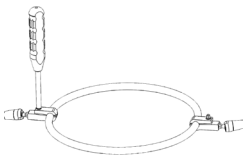
Figure Four
Hex Wrench tightening the ring attachment around the Retractor Ring

Note: Ensure the hex wrench and screw are aligned with the threaded hole. Attempting to insert the screw at an angle may cause cross-threading.

Blade to Blade Holder Assembly:

- Retractor blades have a “dual fixation” connection which allows the blade to remain fixed or to rotate. To attach the **Retractor Blade** to the **Retractor Blade Holder** (AL-0105), insert the blade through the blade holder connection so that the first pin in the connector can pass through the groove in the blade holder.
- When the first pin is engaged with the **Blade Holder**, the **Retractor Blade** is attached and able to rotate.
- If a fixed connection is preferred, the second pin should be aligned with the groove, and the blade pushed upward. When the second pin is engaged in the **Blade Holder**, the **Retractor Blade** will remain fixed (Figure Five).

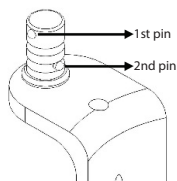
Figure Five
Dual fixation connection

Ring Clamp to Retractor Ring Assembly and Positioning:

- **Ring clamps** (AL-0103) attach to any point of the ring. To attach, loosen knob of the **Ring Clamp** and press the larger of the two openings onto the ring frame; the clamps will snap onto the ring. Press the shaft of the **Blade Holder** into the smaller opening of the **Ring Clamp** to attach the two components. The **Ring Clamp/Blade Holder Assembly** can be freely adjusted until the knob of the clamp is

tightened. Additional **Ring Clamps** and **Blade Holders** may be added to the **Retractor Ring** as desired. (Figure Six).

- To angulate the **Retractor Blades**, use the **Hex Wrench** to loosen the swivel mechanism on the **Blade Holder** and manipulate the blade by hand to the desired position. To secure the blade, tighten the swivel mechanism with the **Hex Wrench** (Figure Seven).

Note: it is only necessary to rotate the mechanism counterclockwise until the swivel mechanism moves freely. To avoid damage to mechanism, do not force screw mechanism past stop.

Lighting System Assembly:

- Loosen the small blue knob of **Light Cable Flex Arm** (HS-0305) and position jaws around the segment of either **Articulating Arm** that is closest to the **Retractor Ring**. Tighten small blue knob to affix the **Light Cable Flex Arm** to the **Articulating Arm**.
- Insert the bifurcated tip of the **Light Cable** (ML-0058 or ML-0048) into the light ports located at proximal end of the **Light Cable Flex Arm**. Ensure that the **Light Cable** is connected to the **LED Light Source** (ML-0051) and that the light source is plugged into a power source. Insert the **Light Cable** into the **LED Light Source**. Turn on the light source to illuminate the operative site.

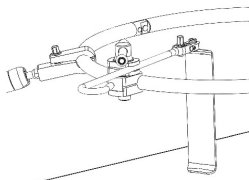
Figure Six
Ring Clamp/Blade Holder Assembly
attached to the Retractor Ring

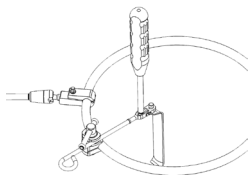
Figure Seven
Use Hex Wrench to loosen swivel
mechanism

- The **Light Cable Flex Arm** may be positioned as desired and readjusted as needed. Tighten the large blue knob to lock the arm into position (Figure Eight).

Note: Attaching the light cable flex arm to the ring segment may obstruct the surgeon's access. For optimal visualization, the Light Cable Flex Arm should be attached onto the Articulating Arm.

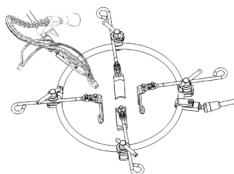
Figure Eight
Full setup with Light Cable Flex Arm

Disassembly:

- Completely loosen the **Articulating Arm** screws, do not over-loosen or force the screw past the stop.
- Disassemble the Retractor Ring by turning the screws counterclockwise twice with the **Hex Tool**.
- It is important that the **Retractor Ring** is disassembled by pulling the rings directly apart on a flat surface.

Note: To prevent damage to the screws and forks on the connection points of the Retractor Ring, pull rings directly apart. Do not cant or twist the rings.

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